Pronto to use. carefully read the instructions for use.

GC FujiCEM™ Evolve

RADIOPAQUE RESIN REINFORCED GLASS IONOMER LUTING CEMENT

For use only by a dental professional in the recommended indications.

INDICATIONS FOR USE

1. Cementation of metal-based inlays, onlays, crowns and bridges
2. Cementation of resin inlays, onlays, crowns and bridges
3. Cementation of all ceramic inlays
4. Cementation of high strength (e.g. zirconia based, lithium disilicate etc.) all ceramic inlays, onlays, crowns and bridges
5. Cementation of metal, ceramic and fibre posts

CONTRAINDICATIONS

1. Pulp capping
2. Avoid use of this product in patients with known allergies to glass ionomer cement, methacrylate monomer or methacrylate polymer.

DIRECTIONS FOR USE

Paste / Paste ratio (g/g)	1.0 / 1.0
Working time (23°C / 73°F) (min., sec.)	2'15"
Finishing time (37°C / 99°F, after setting the restoration) (min., sec.)	4'30"

1. TOOTH PREPARATION

1. Prepare the tooth in a normal manner. For pulp capping, use calcium hydroxide.
 - a) Clean the prepared tooth with pumice and water. Note:
 - 1) NACIO, EDTA or Chlorhexidine can be used to clean the prepared tooth. Do not use substances like H₂O₂ (hydrogen peroxide) and other disinfectants as they may lower the bond strength.
 - 2) Optionally, apply GC Fuji PLUS CONDITIONER on the prepared tooth surface for 20 seconds for optimal adhesion.
2. Rinse thoroughly with water. Remove excess moisture by blotting with a cotton pellet or gently blowing with an oil-free air. DO NOT DESICCATE. Prepared surfaces should appear moist (glistening).
3. DISPENSING
 - A. AUTOMIXING WITH GC Push and Click Tip
 - a) Remove the cap of the syringe. Note:
 - 1) If the material has been refrigerated, leave the syringe in room temperature for 10 minutes.
 - 2) Before every use, check the two openings of the syringe tip and confirm the pastes are at the same level to ensure even flow from the syringe.
 - 3) Before dispensing the first time from a new syringe or after a long interval in between use, bleed the syringe from the syringe. Cut off the extruded paste from the syringe nozzle by moving the syringe to an upright position on a mixing pad (Fig. 1).
 - b) Before attaching the GC Push and Click Tip, confirm the GC Push and Click Tip is seated in the syringe.
 - c) Place GC Push and Click Tip lightly on the syringe nozzle and push the lock unit down to the fixing position until you hear a click (Fig. 3).
 - d) Discard the initial mix until a uniform color palette is reached.
 - e) Extrude the material directly into the restoration.

2. RESTORATION

1. Make sure the restoration is pretreated and handled according to the manufacturers instructions.
3. DISPENSING
 - A. AUTOMIXING WITH GC Push and Click Tip
 - a) Remove the cap of the syringe. Note:
 - 1) Do not leave the used GC Push and Click Tip on the syringe for a long period of time.
 - 2) Store with cap securely attached to the syringe.
 - b. HANDMIXING
 - a) Remove the cap of the syringe. Note:
 - 1) If the material has been refrigerated, leave the syringe in room temperature for 10 minutes.
 - 2) Before dispensing the first time from a new syringe or after a long interval in between use, bleed the pastes from the syringe. Cut off the extruded paste from the syringe nozzle by moving the syringe to an upright position on the mixing pad (Fig. 1).
 - 3) Spread the two pastes thinly on the mixing pad using a plastic spatula. Mix thoroughly, with lapping strokes, for 10 seconds. Be careful not to incorporate air bubbles. Note:
 - 1) Larger amount of paste requires a longer mixing time (up to 15 seconds).
 - 2) Do not mix GC FujiCEM Evolve pastes with other materials.
 - 4) Clean the syringe tip.
 - 5) Replace the cap. Note:
 - 1) Store with cap securely attached to the syringe.

4. CEMENTATION

1. Coat the internal surface of the restoration with sufficient cement and seat immediately on the preparation. The working time is 2 minutes 15 seconds from start of mixing at 23°C (73°F). Higher temperature will shorten and lower temperature will prolong working time. Maintain moderate pressure.
2. Make sure that GC FujiCEM Evolve flows around all margins. Remove excesses using one of the techniques below:
 - a) Keep moderate pressure and make sure the restoration remains in place. Once the excess cement reaches a rubbery consistency, remove it with a probe, a scaler or another appropriate instrument. Make sure that excess cement is thoroughly removed from difficult-to-access regions (e.g. interproximal areas).
 - b) Tack cure each surface with a visible light curing unit for 3 seconds until it reaches a solid rubbery consistency and remove the excess using a probe or other appropriate instrument, while holding the restoration in position. Note:
 - 1) Reaching a good consistency for excess removal depends on the light-curing unit being used. It is important to find the best combination of time, intensity, and distance of the curing light in use.
3. FINAL SET
 - A. Let the material set for 4 minutes 30 seconds after seating the restoration.
7. METHOD. POLISHING
 - A. If needed, finish and polish margins with appropriate instruments.

SHADE

Universal

STORAGE

Recommended for optimal performance, store at room temperature away from heat, moisture and sunlight (4-25°C) (39.2-77.0°F).

PACKAGES

1. GC FujiCEM Evolve Single Pack
 - a) GC FujiCEM Evolve syringe (9.2 g / 5.0 mL) (1), GC Push and Click Tip Regular (45)
2. GC FujiCEM Evolve Triple Pack Automatic
 - a) GC FujiCEM Evolve syringe (9.2 g / 5.0 mL) (3), GC Push and Click Tip Regular (45)
3. GC FujiCEM Evolve Triple Pack Handmix
 - a) GC FujiCEM Evolve syringe (9.2 g / 5.0 mL) (3), Mixing Pad #22 (1)
4. GC Push and Click Tip Regular (15)
5. GC Push and Click Tip Endo (15)

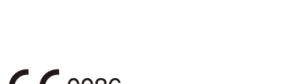
CAUTION

1. In case of contact with oral tissue or skin, remove immediately with a sponge or cotton soaked in alcohol. Flush with water. To avoid contact, a rubber dam and/or cocoa butter can be used to isolate the operation field from the skin or oral tissue.
2. In case of contact with eyes, flush immediately with water and seek medical attention.
3. GC Fuji PLUS CONDITIONER contains 2% Ferric Chloride. It is not recommended to use GC Fuji PLUS CONDITIONER in combination with products containing Epinephrine as this may stain oral tissue.
4. Do not use GC FujiCEM Evolve in combination with eugenol containing materials as eugenol may hinder GC FujiCEM Evolve from setting or bonding properly.
5. Use a protective light shield or similar protective eyewear during light curing.
6. This product is not indicated for filling or core build-up.
7. The GC Push and Click Tip is for single use only. To prevent cross contamination between patients, do not reuse the GC Push and Click Tip. The GC Push and Click Tip can not be sterilized in an autoclave or chemically.
8. In rare cases, the product may cause irritation in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.
9. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be considered as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: <http://www.gceurope.com> or for The Americas: <http://www.gcamerica.com> They can also be obtained from your supplier.

CLEANING AND DISINFECTING: MULTI-USE DELIVERY SYSTEMS: To avoid cross-contamination between patients this device requires midlevel disinfection. Immediately after use inspect device and/or label for deterioration. Do not use if damaged. DO NOT IMMERSE. Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional/national guidelines.

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