VOCO**Amaris® Gingiva****GB**

Instructions for use

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The shade can be individually adapted step by step by mixing the **Opakers**.

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Light-curing:
Für licht-curing die material conventional polymerisation devices are suited. The light output should not be lower than 500 mW/cm².

Application of the Opaker:
Apply some **Opaker** onto the tooth surfaces to be masked and disperse thinly, then light-cure for 40 s. If necessary, apply a second coat of **Opaker** to cover the tooth surface, and light-cure for 40 s. In order to ensure the functionality of the non-dripping syringe do not pull back the plunger during or after use.

Application of Amaris® Gingiva:
Apply fillings of more than 2 mm thickness in layers. Light-cure each layer with blue light for 40 s.

Use transparent strips and light-wedges. Ensure a good marginal seal. Hold the light emission tip of the polymerisation device as close to the surface of the filling as possible. If this distance is more than 5 mm, curing depth may be reduced and limited to the light-cone. Incomplete curing may lead to discoloration and pulpititis-like complaints or loss of the filling. Contouring and finishing of the filling can be conducted immediately after removal of the shaping device, using cooling (e.g. fine or extrafine diamond polishers, polishing disks). Finally, the margin of the filling or preferably the entire tooth should be fluoridated.

Precautions and Indications:
- Side effects are not known. A sensitisation of hypersensitive patients cannot be ruled out.

- If the prescribed application technique cannot be carried out, or in cases of known allergies against any ingredient (Bis-GMA, UDMA, TEGDMA, HEMA, BHT) the **Amaris® Gingiva/Opaker** system must not be used.
- Phenolic substances, especially preparations containing eugenol or thymol inhibit the curing of the restorative. The use of zinc-oxide-eugenol cement or other substances containing eugenol in connection with the restorative composites must therefore be avoided.

- The enclosed application cannulae are for single use and have to be disposed of after use. Cover the syringe with its cap after use.

Storage:
Store at room temperature (4°C – 23°C). Close syringes immediately after dispensing. Do not use **Amaris® Gingiva** after its expiry date.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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