voco**Admira® Fusion**

In accordance with DIN EN ISO 4049

Instructions for use**Product description:**

Admira Fusion is a light-curing, radiopaque nanohybrid ORMOCEER® restorative material. **Admira Fusion** contains 84 % w/w inorganic fillers and is used with a dentine-enamel bond. It is available in syringes and Caps for direct application.

Shades:

A1, A2, A3, A3.5, A4, B1, B2, B3, C2, D3, OA1, OA2, OA3, OA3.5, BL, Incisal, GA3.25, GA5

Indications:

- Class I to V restorations
- Base in class I and II cavities
- Reconstruction of traumatically damaged anterior teeth
- Facetting of discoloured anterior teeth
- Correction of shape and shade for improved aesthetic appearance
- Locking, splinting of loose anterior teeth
- Repairing veneers, small enamel defects and temporary C&B-materials
- Extended fissure sealing
- Restoration of deciduous teeth
- Core build-up
- Composite inlays

Preparation:

Clean the teeth to be treated. If required mark contact points. Allow the material to reach room temperature before use.

Shade selection:

To select a shade, compare the enclosed shade guide with the cleaned tooth before anaesthesia. Moisten the **Admira Fusion** shade sample and tooth and select the shade, preferably in daylight.

Cavity preparation:

In principle, cavity preparation should be carried out according to the rules of adhesive filling therapy and should be minimally invasive to conserve healthy tooth substance. Bevel the enamel margins on anterior teeth. Clean and dry the cavity afterwards. Non-carious cervical lesions do not have to be prepared; thorough cleaning is sufficient here.

Creating a dry working field:

Ensure that the work area is sufficiently dry. Use of a rubber dam is recommended.

Matrix placement:

Use of a shaping aid is recommended for cavities with an approximal portion. Translucent matrices that are wedged in the approximal area are advantageous. Minimal separation facilitates the shaping of the approximal contact and insertion of the matrix.

Pulp protection:

A suitable pulp-protective liner that can be covered with a stable cement, if necessary, should be applied in close proximity to the pulp.

Bond material:

Admira Fusion is used in the adhesive technique with a dentine-enamel bond. All light-curing bond materials can be used. Follow the respective instructions for use with regard to preparation (etching technique) and application.

Application of Admira Fusion:

Apply the selected shade of **Admira Fusion** (see shade selection) in layers that are a maximum of 2 mm thick, adapt with a suitable instrument and light-cure afterwards.

Light-curing:

Conventional polymerisation devices are suitable for light-curing the material. The curing time is as follows when using an LED- / halogen light with a minimum output of 500 mW/cm²:

20 s for shades A1, A2, A3, A3.5, A4, B1, B2, B3, C2, D3, BL, Incisal, GA3.25

40 s for shades OA1, OA2, OA3, OA3.5, GA5

Hold the light emission tip of the device as close as possible to the surface of the filling. Otherwise, the curing depth may be reduced. Incomplete curing may lead to discolouration and discomfort.

Finishing:

The restoration can be finished and polished immediately after removing the matrices (e.g., fine or extra-fine diamond grinder, polisher). The tooth should be fluoridated as a final step.

Admira Fusion can be used for direct and indirect inlays according to customary methods. The physical stability values can be improved by conventional external (additional) curing.

Information/precautionary measures:

- **Admira Fusion** contains methacrylates and BHT. **Admira Fusion** should not be used in case of known hypersensitivities (allergies) to any of these ingredients.
- Phenolic substances, especially preparations containing eugenol or thymol, interfere with the curing of filling composites. Avoid the use of zinc oxide eugenol cements or other eugenol-containing materials in combination with filling composites.
- Each **Admira Fusion** Caps unit should be used for one patient only and should be applied exerting uniform but not excessive pressure.
- Fillings exposed to occlusal forces should be checked at least once a year for early detection of any changes.

Storage:

Store at temperatures of 4 °C to 23 °C. To avoid exposure to light and potentially resulting polymerisation, syringes should be closed immediately after dispensing. Do not use **Admira Fusion** after the expiry date.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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VC 60 AA2750 E1 0115 99 © by VOCO
ORMOCER® is a registered trademark of Fraunhofer Gesellschaft zur Förderung der angewandten Forschung e.V., München, Germany.
VOCO GmbH